SUN

Note from the manufacturer

In accordance with the Personal Protective Equipment Regulation (EU) 2016/425 and REACH Regulation (EC) 2006/1907

The product is made so as to assure adequate protection against solar radiation according to the Standard EN 12312-1:2013.

General information and method of use. Sunglasses for general use. The product is made so as to assure adequate protection against solar radiation according to the Standard EN 12312-1:2013. The protection index is indicated on the sticker affixed to the product. The number indicates the filter category (for the recommended usage see the line of the Table in relation to the number). The model is designed to reduce the effects of solar radiation in sunny and normal use conditions.

Instructions for use. The product must be kept in the appropriate case supplied so that the lenses do not get scratched and to protect the product from impacts. Optimal transport conditions will guarantee you quality solar protection. Cleaning and maintenance. To clean, use liquid detergents and wipe with a soft cloth made of natural fibers. Do not use abrasive products or cloths that could damage the filter characteristics. For maintenance, contact specialized and authorized opticians. Use only replacement parts and original accessories.

Mechanical resistance. Mechanical protection adapted for conditions of normal use during which stresses of strong intensity are not anticipated.

Warning. The product is not adapted for looking directly at the sun. The product is not adapted to exposure to UV rays produced by artificial sources. The product is not adapted to driving in conditions of poor visibility, sunset and night. The products relating to category 4 are not adapted to driving an automobile and are not usable by road users.

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User's Instructions Spectacle Frames

Information

This Spectacle Frame is designed for general use and complies with the requirements of the Medical Device Regulation (EU) 2017/745 and EN-ISO-12870:2018.

Identification information is found on the inside of the Spectacle Frame temple.

The Spectacle Frame is a medical device. When any serious incidents occur in relation to the health of the user, this should be reported to Chrome Hearts as well as the national competent authority in the Member State in which the user is established.

If you experience any physical discomfort, in direct relation with wearing the Spectacle Frame, please consult with a doctor or qualified healthcare professional.

Use

In order to use the Spectacle Frame as intended, the frame shall be used combined with prescription lenses.

Maintenance

Do not use any aggressive cleaning materials to clean our Spectacle Frame.

This Spectacle Frame is sold with a demo lens intended for demonstration purposes only. To address any problems with any lens provided with this Spectacle Frame other than the demo lens, please contact the provider of that lens.

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